WHEELBLADES XL

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MODELLINDEX A

OPERATING MANUAL

MODEL INDEX A

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ISTRUZIONI D'USO

MODELLO A

BRUKSANVISNING

MODELLINDEX A

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11 Kundendienst

WHEELBLADES GmbH
Patrick Mayer
info@wheelblades.com
www.wheelblades.com

WHEELBLADES XL OPERATING INSTRUCTIONS

FOR BABY CARRIAGES, REHAB BUGGIES AND ROLLATORS


MODEL INDEX A, 10/2013

Overview

Wheelblade XL Fully Assembled (attachment mounted on right)

Key:

Pos.	Description	Material
1	Ski	POM (polyoxymethylene)
2	Attachment	POM (polyoxymethylene)
3	Clamping lever	POM glass-fibre reinforced (polyoxymethylene)
4	Try square	POM (polyoxymethylene)
5	Spring plate	Spring steel
6	Fastening screw	Chrome steel
7	Washer	Chrome steel

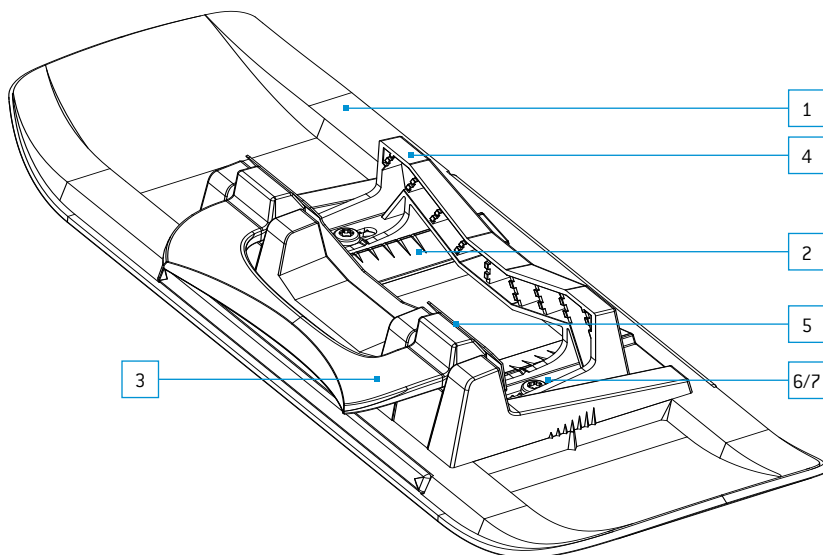Deutsch	4	English	14
Français	24	Italiano	34
Svensk	44	Русский	54

Table of Contents

Overview	14
1 General	16
1.1 Compliance	
1.2 Information on Operating Instructions	
1.3 Explanation of Symbols	
1.4 Scope of Delivery	
1.5 Warranty	
1.6 Service Life	
2 Safety	16
2.1 Proper Use	
2.2 Assistance for Mounting and Dismounting of Wheelblades XL on Rollator	
3 Technical Data	17
3.1 Dimensions and Weight	
3.2 Environmental Conditions	
3.3 Labelling	
4 Design and Function	18
4.1 Clamping Mechanism	
5 Configuration	18
5.1 Initial Configuration	
6 Mounting/Dismounting Instructions	20
6.1 Mounting Instructions	
6.2 Dismounting Instructions	
7 Moving/Travelling with Wheelblades XL	21
8 Cleaning	22
9 Troubleshooting	22
10 Disposal	23
11 Customer Service	23

1 General

1.1 Compliance

Wheelblades XL are in compliance with directive 93/42/EEC for medical devices and directive 2007/47/EC, which contains revisions to the former directive.

WHEELBLADES GmbH
Theaterweg 11
Postfach 27
CH-7002 Chur
Switzerland

1.2 Information on Operating Instructions

In addition to these operating instructions you should also carefully read and observe those of baby carriages, rollators or rehab-buggies to ensure safety while using your Wheelblades XL. The Wheelblades XL are only intended to expand the field of motion on packed snow on streets and pathways.

Use the product only if you have read and understood the operating instructions. Keep this instructional manual in a safe place. Please contact us if you do not understand the mounting instructions or parts thereof, or would like to make suggestions or propose additions. Use the following address:

WHEELBLADES GmbH
Patrick Mayer
info@wheelblades.com
www.wheelblades.com

1.3 Explanation of Symbols

The following symbols alert you to hazards. Failure to observe them may result in serious injuries.

DANGER / CAUTION / WARNING!


These signs indicate a potentially hazardous situation that may result in minor bodily injuries, serious injuries or death if not avoided.

1.4 Scope of Delivery

The following parts are included in the scope of delivery:

- Wheelblades XL, fully assembled
- 1 Torx pin wrench, size T25
- 1 Operating Instructions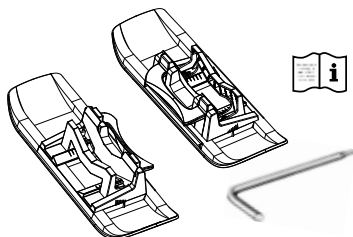

1.5 Warranty

Wheelblades GmbH guarantees the Wheelblades XL to be defect-free and functioning properly. The warranty covers all faults and defects which result from faulty construction, poor choice of materials or defective workmanship. The warranty period is 2 years from date of purchase. The warranty does not cover the following.

- Normal wear and tear
- Improper handling and damage
- Lack of maintenance
- Incorrect mounting
- Defects attributable to external circumstances
- Modifications not expressly approved by Wheelblades GmbH in writing
- Damages due to the use of unsuitable spare parts

The resulting costs are not included in the warranty.

1.6 Service Life

The Wheelblades XL are designed for a service life of approx. 5 years.

2 Safety

2.1 Proper Use

Wheelblades XL are intended for use on packed snow which is free from loose gravel. They are not suitable for travelling on ice, deep snow and snow-free substrata. They are not intended for use on ski or toboggan runs and the like.

- Wheelblades XL serve to expand the field of motion of baby carriages, rollators and rehab-buggies. They are not sporting equipment and not designed for forces experienced in sporting activities.
- Wheelblades XL should be used only for hand-driven baby carriages, rollators and rehab-buggies.
- Wheelblades XL are not suitable for moving on steps or stairs.
- Wheelblades XL should be mounted only on the front wheels. They are designed for wheel diameters of 150 mm to 300 mm and widths of 20 mm to 70 mm.
- The use of the Wheelblades XL must not impair the functional efficiency of the parking brakes of baby carriages, rehab buggies and rollators. Mount only on wheels not affected by the parking brakes.
- Wheelblades XL should be mounted and used only by adults.
- The users must be in full possession of their mental and physical faculties and have read and understood the operating instructions.

DANGER!

Failure to observe proper use may result in hazardous situations which can lead to serious injuries or death.

DANGER!

Choking! The product contains various small parts. Please keep in a safe place (incl. tools) out of reach of children.


2.2 Assistance for Mounting and Dismounting of Wheelblades XL on Rollator

Seek assistance for mounting and dismounting of Wheelblades XL on steering wheels of rollator in order to prevent it from tipping over.

3 Technical Data

3.1 Dimensions and Weight

Fig. 1: Dimensions

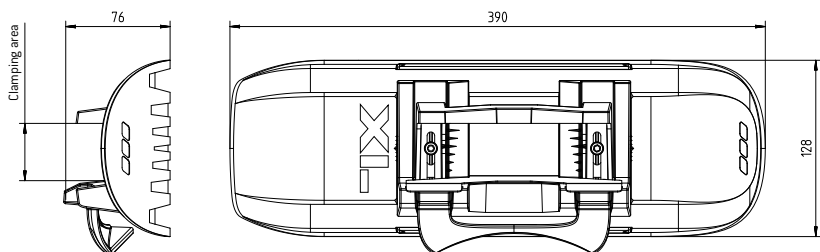Clamping area for wheel widths
Clamping area for wheel diameter
Weight per Wheelblade XL

20 – 70 (mm)
150 – 300 (mm)
0.65 (kg)

3.2 Environmental Conditions

Temperature

–20 bis +30 (°C)

3.3 Labelling

For clear identification, each Wheelblade XL comes with a nameplate indicating model index and serial number. Make sure that the operating instructions match the model index on your Wheelblade XL.

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4 Design and Function

4.1 Clamping Mechanism

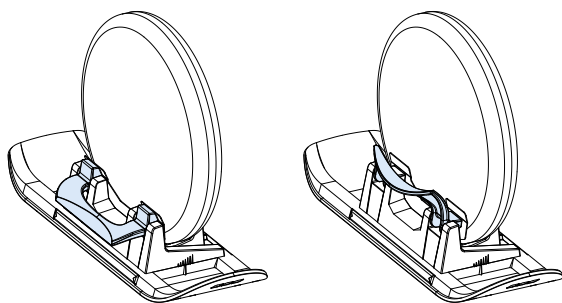
Fig. 4: Clamping mechanism open Clamping mechanism closed

5 Configuration

The Wheelblades XL should be mounted only on the front wheels of baby carriages, rehab-buggies and rollators. They are designed for wheel diameters of 150 mm to 300 mm and widths of 20 mm to 70 mm.


The use of Wheelblades XL must not impair the functional efficiency of the parking brakes of baby carriage, rehab-buggies and rollators. Mount only on wheels not affected by the parking brakes.

In the event that Wheelblades XL have not been adjusted to your baby carriage, rehab buggy or rollator by the manufacturer or the vendor, you must configure them yourself.

5.1 Initial Configuration

2 scales are used for setting the clamping width. An outer scale, used for setting the position of the attachment and the ski; and an inner scale, used for setting the position of the try square and the attachment.


Fig. 5: Setting the scales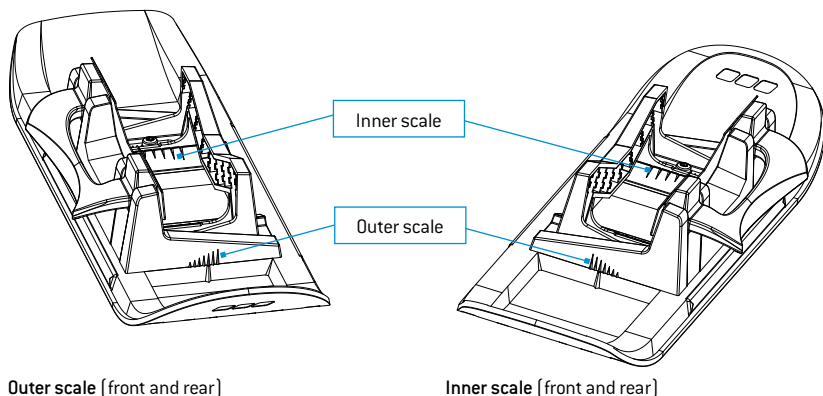

Outer scale (front and rear)


Inner scale (front and rear)

Fig. 6: Setting the Scales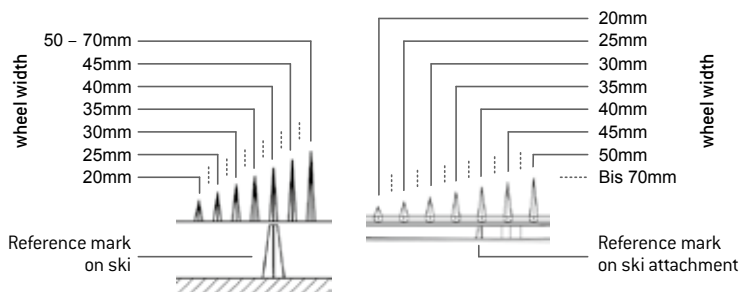

- 1) Loosen both fastening screws so that the attachment no longer engages the ski and can be moved freely.
- 2) Measure the width of the wheels. (e.g.: wheel width 31.5mm)
- 3) Round off the value to the nearest millimetre (e.g.: wheel width 32mm)
- 4) Place the attachment on the ski so that the rounded value approximately corresponds to the outer scale. Orient yourself to the enlarging triangles of the scale and in accordance with Fig. 6
- 5) Make sure that the ski is mounted straight. Use the same setting on both outer scales.
- 6) Proceed in the same manner for setting the try square. In doing so, orient yourself to the two inner scales of the attachment.
- 7) Tighten up the screws.


e.g.: wheel width 32mm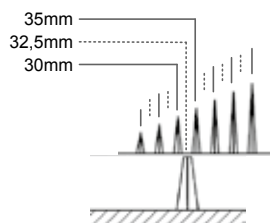

Intermediate positions (grids) are also possible between the attachment and ski.

Fig. 7: Setting the outer scale

e.g.: wheel width 32mm

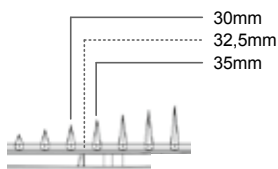From a wheel width of 50 mm, the wheel can no longer be fixed centrally

Fig. 8: Setting the inner scale

- 8) Check setting by inserting the wheel between the try square and spring plate and pulling the clamping lever by 90 degrees upwards. The Wheelblade XL must now sit firmly on the wheel. If it can be pulled off of the wheel with little effort, loosen the fastening screws a little, so that the try square can be moved. Move the try square inwards by a grid and tighten the fastening screws. Repeat step 8 until the Wheelblade XL sits firmly on the wheel.
- 9) Repeat steps 1 through 8 for adjusting the second Wheelblade XL.

DANGER!

Fall! If Wheelblades XL are configured incorrectly or if the screws are loose, the Wheelblades XL may fall off during use.

→ Perform the configuration correctly.

DANGER!

Choking! The product contains various small parts. Please configure the Wheelblades XL out of the reach of children.

6 Mounting/Dismounting Instructions

Before initial use, Wheelblades XL need to be adjusted to the front wheel of your baby carriage, rollator or rehab-buggy. Proceed as described in section 5 (Configuration).

DANGER!

Fall! Defects in Wheelblades XL can result in a fall.

→ Before each use, check Wheelblades XL for damages.

**CAUTION!**

Crushing of fingers! Fingers might get trapped between clamping bracket and ski as well as between the clamping bracket and the front wheel of the baby carriage, rollator or rehab-buggy.

- Do not hold fingers between the clamping bracket and ski during mounting/dismounting.
- Do not hold fingers between the clamping bracket and the front wheel during mounting/dismounting.

6.1 Mounting Instructions

- 1) Check to make sure that fastening screws have been tightened firmly on Wheelblades XL. Retighten them if necessary.
- 2) Lock the baby carriage, rehab buggy or rollator using the parking break.
- 3) Secure the front wheels of your baby carriage, rehab buggy or rollator (in case of rollators, only if possible) to cancel the swivel function. For doing so, consult the operating instructions of the baby carriage, or rollator.
- 4) Make sure there are no objects or snow in the recess provided in Wheelblades XL.
- 5) Place the "right" Wheelblade XL on the "right" front wheel. Make sure that the clamping lever is facing outwards (locking mechanism open) and the ski tip is faces forwards (Wheelblade XL in moving direction).
- 6) Place the respective wheel between the try square and spring plate, in the recess provided in the Wheelblade XL.

Fig. 9

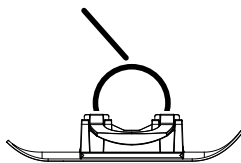
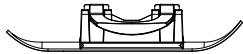
Fig. 10

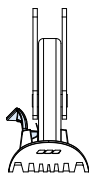
Fig. 11

- 7) When the wheel is at the deepest position in the recess, fold the clamping bracket by 90° upwards until it engages and can no longer be moved further.
- 8) Check to see if the Wheelblade XL is firmly clamped onto the wheel.
- 9) Repeat steps 1 through 6 for the second "left" Wheelblade XL

DANGER!

Fall! Child falls out of the baby carriage

During mounting/dismounting of the Wheelblade XL the baby carriage, rehab buggy can tip over or start moving.

- Lock the baby carriage, rehab buggy or.
- Take baby out of the baby carriage, rehab buggy or secure the child by fastening the seat belt.

DANGER!

Fall! Losing the balance

During mounting/dismounting of the Wheelblade XL the baby carriage, rehab buggy can tip over or start moving.

→ Lock the rollator.

→ Seek the help of an assistant.

**DANGER!**

Fall!

Incorrect mounting or loosened screws.

→ Perform the mounting/basic setting correctly.

**DANGER!**

Fall!

Jack-knifing of the Wheelblade XL

→ Use the Wheelblade XL only with reversed swivel wheel function of the front wheels of baby carriages, rollators or rehab-buggies.

6.2 Dismounting Instructions

- 1) Lock the baby carriage, rehab buggy, rollator
- 2) Fold the clamping bracket of the Wheelblade XL downwards (clamping mechanism open)
- 3) Lift the front wheels out of the Wheelblade XL.

DANGER!

Fall! Child falls out of the baby carriage, rehab-buggy, rollator.

During mounting/dismounting of the Wheelblade XL the baby carriage, rehab buggy can tip over or start moving.

→ Lock the baby carriage, rehab-buggy.

→ Take child out of the baby carriage, rehab buggy or secure the child by fastening the seat belt.

**DANGER!**

Fall! Losing the balance

During mounting/dismounting of the Wheelblade XL, the baby carriage, rehab buggy can tip over or start moving.

→ Lock the rollator.

→ Seek the help of an assistant.

7 Moving/Travelling with Wheelblades XL

The driving behaviour differs from the common driving with rollers on asphalt. Therefore, you should first carefully test-drive on a secure terrain.

Pay attention to the instructions for use contained in the operating manual of your baby carriage, rehab-buggy, or rollator. The Wheelblades XL are intended only as an expansion of the range of motion. Move the baby carriage, rehab buggy or rollator always with caution and use the Wheelblades XL only on streets and pathways with packed snow.

WARNING!

Make sure all locks are engaged before use that.

**WARNING!**

Do not allow your child to play with the baby carriage or rehab buggy unattended.

DANGER!

Fall! Uncontrolled/accidental driving off of the baby carriage, rehab-buggy.

- Never leave the baby carriage, rehab buggy and your child unattended.
- Always lock the parking brake of the baby carriage, rehab buggy or secure your baby carriage, rehab buggy against uncontrolled/accidental driving off.

**DANGER!**

Never exceed a speed of 5 km/h. The Wheelblades XL and the baby carriage, rehab buggy are not designed to be used for jogging, skating, for cross country skiing, or for performing other snow sports.

**DANGER!**

Fall! Objects caught between the clamping bracket and the steering wheel or between clamping bracket and steering shaft can open the clamping mechanism and cause the Wheelblade XL to fall off.

- Make sure no objects (branches, etc.) are caught between the clamping bracket and the steering wheel or between the clamping bracket and the steering shaft.

**DANGER!**

Fall! Sinking and jack-knifing of the Wheelblade XL in loose or deep snow.

- Avoid areas with loose or deep snow.

**DANGER!**

Fall! Abrupt changes of the subsurface result in high resistance (e.g.: asphalt, gravel).

- Avoid areas with low or no pressed snow.

**DANGER!**

Collision/Fall! Loss of control on ice.

- Never use the Wheelblade XL on ice.

8 Cleaning

Clean the Wheelblades XL regularly. Only then can the proper function be ensured.

Rinse the Wheelblade XL briefly with lukewarm water and let dry.

Do not put Wheelblade XL in the dishwasher!

Avoid contact with aggressive solvents.

9 Troubleshooting

The wheel does not clamp properly:

- Check the adjustable clamping width (see Section 5)
- Parts are not mounted straight during configuration.
- Free the recess provided in Wheelblade XL from any objects or snow.

10 Disposal

No longer used Wheelblades XL can be completely dismantled into their component parts. For dismantling remove the two fastening screws. The Wheelblade XL can be recycled in the usual locations for disposal of plastic and metal. The material type for each item is listed in the Overview.

CAUTION!

Cuts! Sharp edges.

→ Be careful of sharp edges when dismantling.

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